

## The original MEYLAND CARLIFT



*The permanent lift console is the only visible part in the car when the lift is not in place.*

### **The MEYLAND CARLIFT is the perfect lifting solution in the car!**

The MEYLAND CARLIFT lifts disabled persons in and out of cars without problems. The lift is operated either by the disabled person, or - if necessary - by an assistant. The MEYLAND CARLIFT can be mounted in either the left or the right hand side of cars with two or four doors, and the lift arm is adjustable to fit the lift to different car sizes. The lifting/lowering function works by a 12 volt motor, which is supplied with power from the car battery. The lift is operated from a small control box with push buttons. The box is connected to the lift with a spiral cable long enough to make it possible for an assistant outside the car to operate the lift. The sideways movements are done manually. The MEYLAND CARLIFT is supplied with a nylon sling seat which has three mounting points to give a safe lift, and the sling seat is available in 2 different sizes (see backside). When the MEYLAND CARLIFT is not in use or while driving it can either stay on the console or in one single grip be lifted off and put into the trunk. If the lift console is removed no visible traces are normally seen, which is an advantage if the car is to be sold.


### **The MEYLAND CARLIFT is the perfect lifting solution at home!**

The MEYLAND CARLIFT can also be used at home in the bedroom, the bathroom etc. and is operated in the same way as in the car. The lift can be used for lifting the person from the wheelchair to the bed, the toilet, the bath etc. Furthermore, the MEYLAND CARLIFT can lift a person off the floor! For the use at home a height adjustable wall console is available, which is bolted onto the wall at the desired place. The MEYLAND CARLIFT is lifted onto the console and supplied with power from a 12/230 volt transformer. Both lift and transformer can be used elsewhere in the house wherever wall consoles are mounted. An extension arm is available making the lift cover a wider area. The extension arm is also useful where the space is limited.

The MEYLAND CARLIFT is lifted off the console in one single grip – weighs only 13,5 kg (30 lbs.) – but lifts up to 125 kg (275 lbs.)!



## Sling seat for the MEYLAND CARLIFT

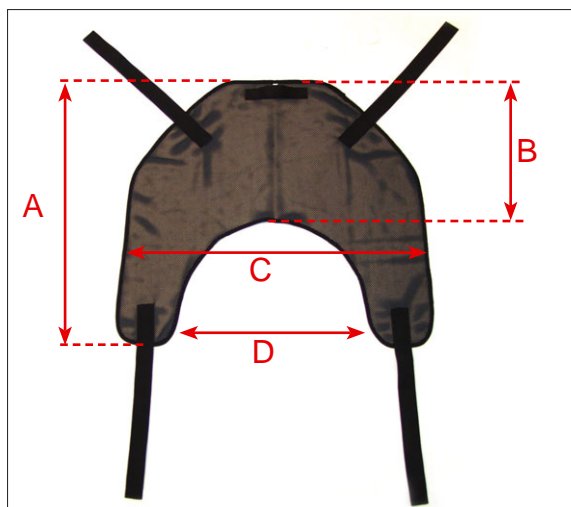

## The sling seat is available in 2 sizes:

Sling seat:	A:	B:	C:	D:
No. 1	102 cm (40")	46 cm (18")	91 cm (36")	56 cm (22")
Size corresponds approx. to a bodyweight of 70 - 125 kg and a height of 1.70 m or more.				
No. 5 (std.)	83 cm (32,5")	46 cm (18")	86 cm (34")	50 cm (19,5")
Size corresponds approx. to a bodyweight of up to 70 kg and a height of less than 1.75 m.				

## Technical data

Length:	57 - 65 cm	(22,5" - 25,5")
Width:	10 cm	(4")
Sling seat, standard:	No. 5	
Min. height:	35 cm	(13,5")
Lifting height:	39 cm	(15,5")
Max. load:	125 kg	(275 lbs.)
Weight:	13,5 kg	(30 lbs.)

## Colour



Gray (RAL 7011)



Nylon net

## Construction

- Manufactured from steel
  - Powder-coated / electrogalvanised
  - Special quality actuator from SKF
  - Relay controlled with built-in error security
  - Sling seat in black net with broad nylon webbings
- (We reserve the right to make changes to design and construction.)

## Accessories


- Extra lift console mounting set for a car
- Wall console
- Extension arm
- Height increasing pin
- Extra canvas sling seat

Dealer:

Manufacturer:

**meyland-smith**  
mobility and more

Industrivej 27  
DK-9830 Taars  
Tel: +45 98 96 19 85  
Fax: +45 98 96 19 86  
[www.meyland-smith.dk](http://www.meyland-smith.dk)

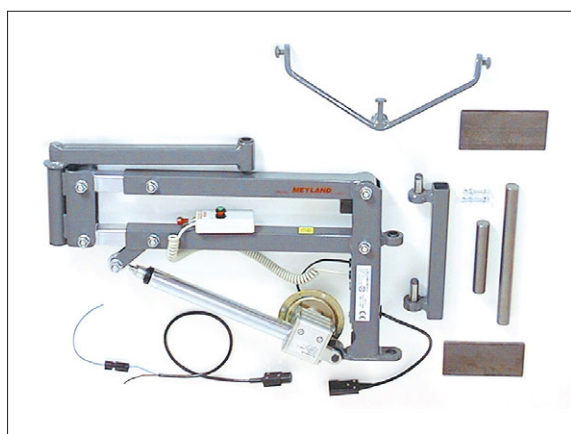

All the parts for mounting the lift in a car.